

# OSOM<sup>®</sup>

## Influenza A&B Test

*When rapid, differential diagnosis is essential  
to your patient's health and comfort.*



✓ **ACCURATE, DIFFERENTIAL DIAGNOSIS**

Differentiates between influenza A & B.  
Three distinct lines.

Capable of detecting some animal and avian  
influenza strains.\*

✓ **QC INSIDE**

Includes 2 extra test sticks for external QC testing.

✓ **NASAL SWAB SPECIMENS**

✓ **FAST**


Less than 1 minute hands on time.  
Results in 10 minutes.

✓ **ROOM TEMPERATURE STORAGE**




**TEST PROCEDURE / RESULTS**

Add  
extraction  
buffer to  
the tube.

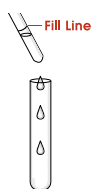

Add swab  
to tube and  
vigorously mix  
by rotating at  
least 10 times.

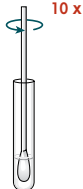


Squeeze  
side of tube  
to express  
as much  
liquid as  
possible from  
the swab.



Add test  
stick to tube.  
Read results  
at 10 minutes.



10 MIN

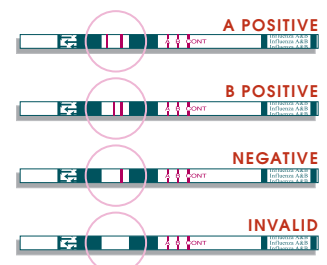

FOR MORE INFORMATION ON OUR RAPID TEST PRODUCTS, VISIT US AT [www.genzymediagnosics.com](http://www.genzymediagnosics.com)

**ORDER:**

#190, OSOM<sup>®</sup> Influenza A&B Test, 25 Tests

#191, OSOM<sup>®</sup> Influenza A&B Control Kit, 5 A&B Swabs

#190E, OSOM<sup>®</sup> Influenza A&B Test, 25 Tests (CE marked)

**The Americas**

One Kendall Square  
Cambridge, MA 02139-1562, USA  
Customer/Technical Service: 800-332-1042  
E-mail: [usdiagcustomerservice@genzyme.com](mailto:usdiagcustomerservice@genzyme.com)

ISO 13485 Certified

**International**

50 Gibson Drive  
Kings Hill, West Malling  
Kent, ME19 4AF, UK  
Tel: 44 (0) 1732-220022  
Fax: 44 (0) 1732-220024  
E-mail: [ukdiagcustomerservice@genzyme.com](mailto:ukdiagcustomerservice@genzyme.com)

ISO 9001:2000, ISO 13485

\* Although this test has been shown to detect cultured avian influenza viruses, including avian influenza A subtype H5N1 virus, the performance characteristics of this test with specimens from humans infected with H5N1 or other avian influenza viruses are unknown. See package insert for details.