


Transfer Tubes

A special plastic material permits fluids to be automatically drawn into the tube by capillary action. A micro hole on the side wall of the tube allows venting and filling volume control.


Reorder
Catalog Numbers
202012 (12 μ L)*

* Provided in bags
of 100 Tubes

Air hole regulates how much tube will fill.

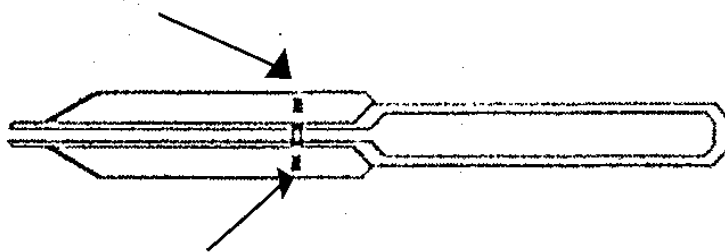

Tube will fill to line automatically. To collect correct volume, sample must fill to line.

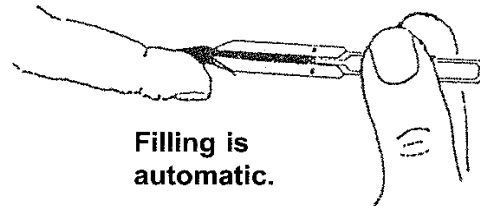


DO NOT squeeze tube while filling.

Filling is automatic.

While holding tube horizontally, touch the tip to the blood sample. Capillary action will automatically draw the sample to the fill line and stop.

Place the tip of the tube over the center of the Test Card and squeeze the tube to expel the sample "all at once."

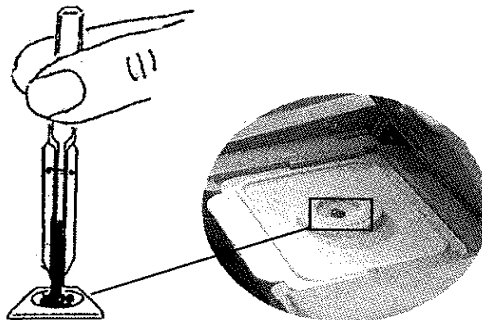

Do not reuse sample

CLIAwaived, Inc.

Tel: (858) 481-5031

web site: www.CLIAwaived.com