

CLIAwaived™ Inc. Rapid Drug Test Cup

NEW!



The CLIAwaived Inc. Rapid Drug Test Cup is one of the simplest, easiest-to-use drugs of abuse test in the market. It provides fast, and accurate results in minutes.



Just turn and rotate the cup to activate.

The all new CLIAwaived™ Inc. RDTC Rapid Drug Test Cup is the first ever FDA-Cleared and CLIA Waived 12 panel drug test cup. Exclusively marketed by CLIAwaived.com. Features a simple design for simultaneous evaluation and detection of 12 drugs of abuse in urine; including THC, COC, OPI, AMP, mAMP, PCP, BAR, BZD, MTD, MDMA, TCA and OXY. (Note: No adulterant test strips integrated in the cup.)

BENEFITS:

- Simple procedure
- Easy to interpret color bands
- Built-in test control
- Room temperature storage
- Read results in 5 minutes
- FDA-Cleared / CLIA Waived for up to 12 drugs

Ordering Information:

Product Description	Catalog #
CLIAwaived™ Inc. Rapid Drug Test Cup (25 cups) (THC, COC, OPI, AMP, mAMP, PCP, BAR, BZD, MTD, MDMA, TCA and OXY - No adulterant test strips)	CLIA-12 RDTC_02

Reimbursement Information:


CPT Code #	National Limit Amount
Effective 4/1/2010, the Healthcare Common Procedure Coding System (HCPCS) included the following new codes: G0430 – Drug screen, qualitative; multiple drug classes other than chromatographic method, each procedure G0431 – Drug screen, qualitative; single drug class method (e.g., immunoassay, enzyme assay), each drug class.	\$20.83

DISCLAIMER – The CPT information contained in this document is provided as representative examples of reimbursement in this category. It is intended to assist providers in accurately obtaining reimbursement for health care services. It is not intended to increase or maximize reimbursement by any payor. Providers should consult their payor organizations with regard to local reimbursement policies. The information provided in this document is for information purposes only and represents no statement, promise or guarantee by CLIAwaived™ Inc. All CPT codes are supplied for information purposes only and represent no statement, promise or guarantee by CLIAwaived™ Inc. that these codes will be appropriate or that reimbursement will be made. CPT codes and descriptions are copyrights of the American Medical Associations. CPT does not include fee schedules, relative values or related ratings. The source for this information is the Center for Medicare and Medicaid Services. The content provided by the Center for Medicare and Medicaid Services is updated frequently. It is the responsibility of the service provider to confirm the appropriate coding required by their local Medicare carriers, fiscal intermediaries and commercial payors. All product information and prices are subject to change without notice at any time. Please call our office before ordering to verify current prices.

CLIAwaived™ Inc. Rapid Drug Test Cup Instructions

TEST PROCEDURE:

1. Urinate into the cup, filling the cup with urine above the minimum level line.
2. Secure the test lid to the specimen cup.
3. Tip the cup and rotate to activate the test.
4. Wait 5 minutes.
5. Read the results.
5. Refer to product insert for detailed instructions.

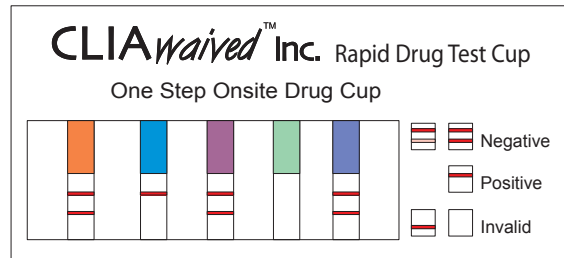

- **ACCURACY:** FDA 510(k) registered detection strips have the same high degree of accuracy when compared to SAMHSA certified laboratory screening methods.
- **SELF-CONTAINED:** Each cup includes a temperature strip and identification label. Each cup is sealed in its own bag and has an 14-16 month shelf life.
- **NO LEAKAGE:** The cup does not leak so a positive specimen can be shipped in the same container to a laboratory for GC/MS confirmation.
- **COST:** In addition to providing immediate results, the CLIAwaived Inc. Rapid Drug Test Cup costs less than half the price of laboratory testing.

Rev. 03.26.10