

BACtrack® Select S30 Breathalyzer Test

Distributed by

CLIAwaived™ Inc.



The BACtrack Select Breathalyzer S-30 is an easy to use device that tests the blood alcohol content and delivers a quick and accurate result. With the versatile breath intake area, users can use the BACtrack with or without the replaceable, hygienic mouthpieces that are included. The simple to read, 3 digit display provides users with easy to read test results and alerts, while the compact size allows users to keep this breathalyzer in their pocket or purse. With the one button operation, users can simply turn the device on and begin testing within seconds.

BENEFITS:

- Simple to use breath intake area
- Versatile design allows for quick use
- 3 digit multifunction display
- Easy to read test results and alerts
- One button operation
- Pocket size compact design



Ordering Information:	
Product Description	Catalog #
BACtrack® Select S30 Breathalyzer Test (Breathalyzer, 2-mouthpieces, 1 mouthpiece adaptor, 2-“AAA” batteries)	BAC-S30
BACtrack® Select Series Mouthpieces (10 pack)	BAC-MPS-10
BACtrack® Select Series Mouthpieces (50 pack)	BAC-MPS-50
Reimbursement Information:	
CPT Code #	National Limit Amount
N/A	N/A

All product information and prices are subject to change without notice at any time. Please call our office before ordering to verify current prices.

BACtrack® Select S30 Breathalyzer Test Specifications

SPECIFICATIONS

Dimensions	1.33 x 4.05 x 0.98 inches (3.4 x 10.3 x 2.5 cm)
Weight	1.87 oz (53 g)
Sensor Technology	Advanced semiconductor
Detection Range	0.00 – 0.40 %BAC
Test Count	Displays total number of tests performed
Power Supply	Two AAA alkaline batteries, included
Battery Life	Approximately 200~300 times
Warm Up Time	10-20 seconds
Response Time	5 seconds
Sensor Accuracy	+/- 0.01 %BAC @ 0.02 %BAC
Operating Temperature	50-104 °F (10-40 °C)

This device has been tested using NHTSA/DOT procedures. In this testing, the accuracy of this device was established at Blood Alcohol Concentrations (BAC) of 0.008 and 0.032, but not at greater concentrations. Additional laboratory testing has been performed at concentrations ranging from 0.00 – 0.40% BAC.

Warranty: Limited 1 year warranty on labor and parts

Return policy: Due to hygiene purposes, this item is non-returnable.

PRECAUTIONS

1. Wait at least 20 minutes after drinking, eating or smoking before testing. Not observing this waiting period can cause inaccurate readings and damage the sensor.
2. Do not blow smoke, food, or liquids into the BACTRACK as this will damage the sensor.
3. Do not test in areas with strong winds, smoke, or in an area where a large amount of alcohol is being consumed.
4. Replace the two AAA batteries when the battery indicator icon reaches one bar.
5. Send your tester in for periodic calibration service as required. (See calibration).
6. The BACTRACK Breathalyzer is designed to be used in a temperature range of 50-104 °F (10-40 °C). Operation of the unit in temperature ranges above or below this range may affect the accuracy of results.
7. Avoid testing in the presence of any substances that contain methyl alcohol, isopropyl alcohol or acetone. These substances may interfere with the results of the test.
8. You cannot use the results of this product in court.
9. Do not use the BACTRACK as a tool to determine whether you should operate a motor vehicle or equipment, or perform any other dangerous act.
10. Do not drink and drive. Always have a designated driver when alcohol is being consumed.