

# CLIA - 10 URINE REAGENT STRIPS 100 STRIPS - 10 PARAMETERS

FOR IN VITRO DIAGNOSTIC USE  
CLIAwaived, Inc.

TESTS AND READING TIME	1	2	3	4	5	6
<b>LEUKOCYTES</b> 2 min.	NEGATIVE 0	TRACE 15	SMALL (+) 70	MODERATE (++) 125	LARGE (+++) 500 ca CELLS/ $\mu$ L	
<b>NITRITE</b> 60 sec.	NEGATIVE	POSITIVE (ANY DEGREE OF PINK COLOR)				
<b>URO-BILINOGEN</b> 60 sec.	0.2 NORMAL 3.2	1 16	2 32	4 64	8 128 EHRlich UNITS/dl URINE $\mu$ mol/L	
<b>PROTEIN</b> 60 sec.	NEGATIVE	TRACE	30 (+) 0.3	100 (++) 1.0	300 (+++) 3.0	>2000 (++++) >20 mg/dl g/L
<b>PH</b> 60 sec.	5.0	6.0	6.5	7.0	7.5	8.0
<b>BLOOD</b> 60 sec.	NEGATIVE 0	NON-HEMOLYZED 10 TRACE	HEMOLYZED TRACE	SMALL (+) 25	MODERATE (++) 80	LARGE (++) 200 ca CELLS/ $\mu$ L
<b>SPECIFIC GRAVITY</b> 45 sec.	1.000	1.005	1.010	1.015	1.020	1.025
<b>KETONE</b> 40 sec.	NEGATIVE	TRACE (5) 0.5	SMALL (15) 1.5	MODERATE (40) 4.0	LARGE (80) 8.0	LARGE (160) 16 mg/dl mmol/L
<b>BILIRUBIN</b> 30 sec.	NEGATIVE	SMALL +	MODERATE ++	LARGE +++		
<b>GLUCOSE</b> 30 sec.	NEGATIVE	100 5	250 (+) 15	500 (++) 30	1000 (++) 60	>2000 (++) 110 mg/dl mmol/L



**IMPORTANT: READ PACKAGE INSERT BEFORE USE.**  
DO NOT touch test areas. Store at temperature between 15°-30° (59°-86°F). DO NOT remove desiccants. Be sure to replace cap immediately and tightly. V/B

Distributed by: **CLIAwaived.net**  
San Diego, CA 92121  
tel 858-481-5031  
toll free 888-882-7739  
www.cliawaived.net

DATE OPENED \_\_\_\_\_ DO NOT USE AFTER 90 DAYS OF BREAKING THE FOIL SEAL