

The Only One-Step On-Site Oral Fluid Device

Oratect® II Oral Fluid Drug Screen Device


Drugs of Abuse Test Cut-off Levels*:

Methamphetamines** (ME)	25 ng/mL
Marijuana*** (TH)	40 ng/mL
Cocaine (CO)	20 ng/mL
Amphetamines (AM)	25 ng/mL
Opiates (OP)	10 ng/mL
Phencyclidine (PC) or	4 ng/mL
Benzodiazepines (BZ)	5 ng/mL

*Concentration in undiluted neat oral fluid

**Includes Ecstasy (MDMA) at 25 ng/mL

***Parent THC

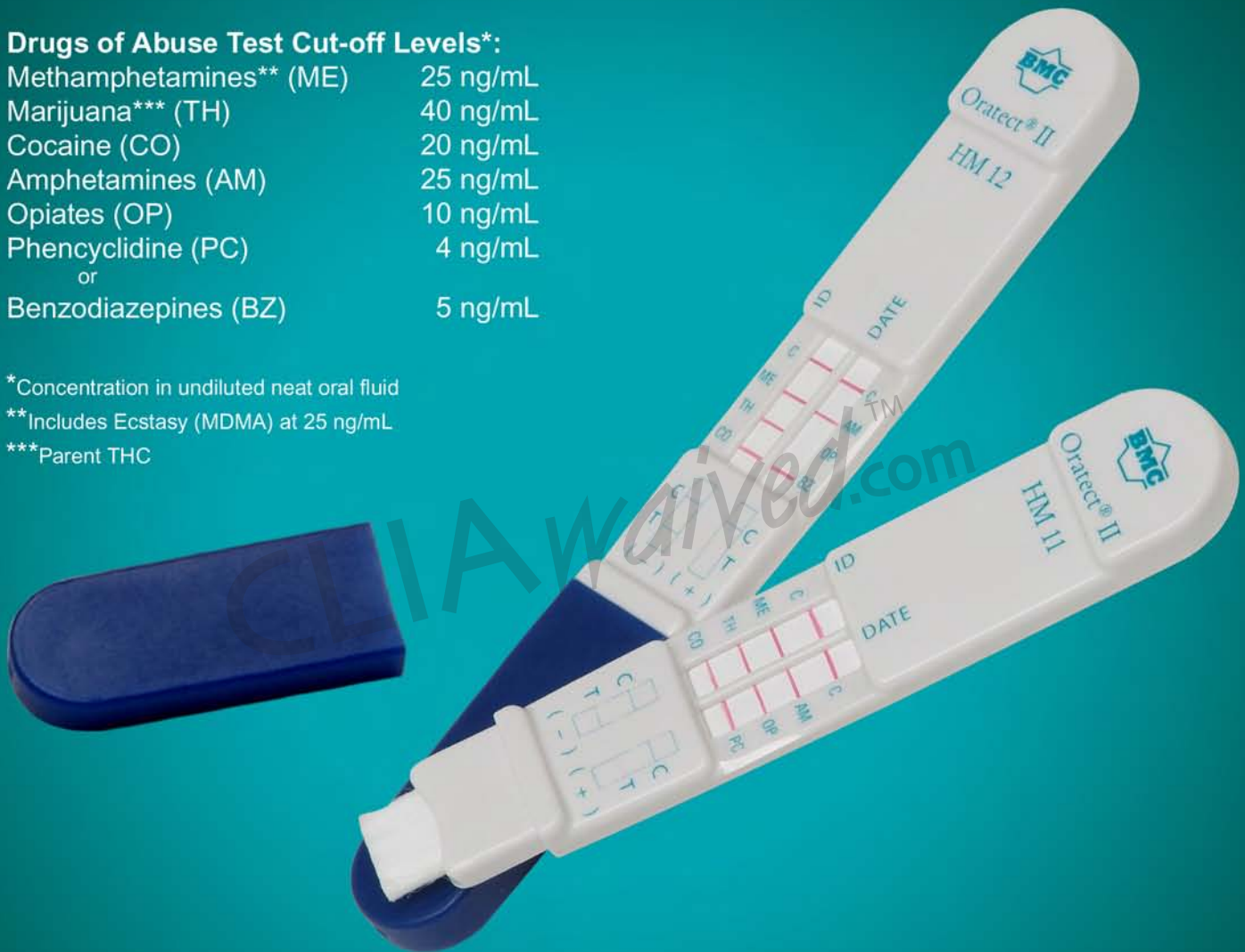

- Collection and testing in one integrated device
- Increased sensitivity of drug cut-off levels
- Two test configurations of 6 drugs
- Donor performed test with minimal administrator supervision
- Rapid collection and testing; results in 5 min
- User-friendly and non-invasive
- Avoids gender observation
- Eliminates sample adulteration or contamination
- Built-in confirmation for chain-of-custody testing








Oratect® II Reference Guide

The Oratect® II* Oral Fluid Drug Screen Device is a simple one-step test for the detection of drugs of abuse in oral fluid.

Oratect® II Procedures -- Refer to package insert for detailed instructions and technical information.

<p>1. Remove the blue cap by holding the sides and pulling gently. This will expose the collection pad. Do not touch the test window area.</p> 	<p>2. Open mouth and rub the collection pad inside mouth against cheek in a circular motion twenty times. Make sure to keep head level.</p> 	<p>3. Rub the collection pad against the opposite cheek in circular motion twenty times.</p> 
<p>4. Rub the collection pad on top of the tongue twenty times. Do not chew, suck, bite or bend the collection pad.</p> 	<p>6. Place the collection pad underneath the tongue to collect saliva. Instruct the donor to hold the device in place with hand. If no pink-purple flow appears (indicated by arrows) in the test windows after 30 seconds, repeat steps 2-6.</p>  <p style="text-align: right;">Test Windows</p>	<p>7. Remove from mouth as soon as pink-purple flow patterns appear at both of the test windows. Cap the device with the blue cap.</p>  <p>8. Lay the device on a flat surface and read results in 5 minutes after removing the device from mouth. Do not read results after 7 minutes.</p>

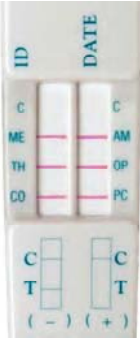Interpreting Oratect® II Test Results

<p>Invalid Result</p> <p>When no band appears in the CONTROL (C) region, the test is invalid even if there is a band in the test region. Repeat the test with a new device.</p>  <p>Example Interpretation:</p> <p>ME: Invalid TH: Invalid CO: Invalid AM: Invalid OP: Invalid PC: Invalid</p>	<p>Negative Result</p> <p>For each test, two colored bands should be observed:</p> <ul style="list-style-type: none"> • One in the CONTROL (C) region • One in the specific TEST region <p>The color of the test band may be slightly darker or lighter than the control band. Any visible band that can be seen is a negative result.</p>  <p>Example Interpretation:</p> <p>ME: Negative TH: Negative CO: Negative AM: Negative OP: Negative PC: Negative</p>	<p>Presumptive Positive Result</p> <p>A colored band at the CONTROL (C) region should be observed. When there is no colored band at the specific TEST region, the test is presumptive positive for that particular drug.</p>  <p>Example Interpretation:</p> <p>ME: Positive TH: Negative CO: Negative AM: Negative OP: Negative PC: Negative</p>
--	--	--